


# SMI-SPON

ABSORBABLE GELATIN SPONGE

- ✓ *Secure and quick haemostasis, stops bleeding fast*
- ✓ *Easy to cut according to the needs*
- ✓ *Biocompatible and non pyrogenic*
- ✓ *Totally absorbed in 3-4 weeks*
- ✓ *Sterile, individually blister packed and ready to use*

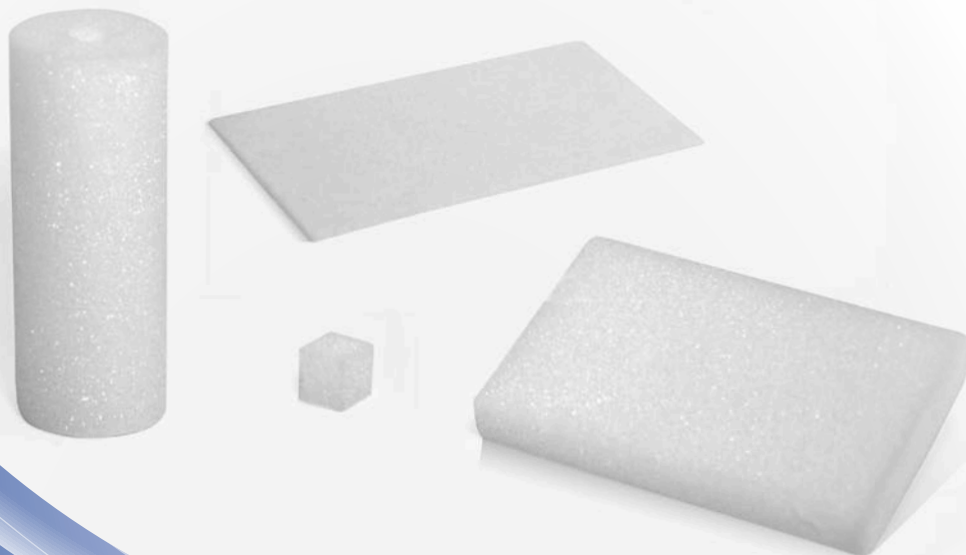

*For quality  
&  
safety*

**smi**

[www.sutures.be](http://www.sutures.be)

# SMI-SPON

ABSORBABLE GELATIN SPONGE

## What is SMI SPON

SMI-SPON is a surgical haemostatic sponge, manufactured from highly purified first grade gelatin material for use in various surgical procedures. It is a quick and effective haemostatic which stops bleeding fast. It reduces intraoperative bleeding and surgery time.

## Why using SMI-SPON

SMI-SPON can be used in various surgeries for haemostasis. When applied in vivo and used in appropriate amounts, it is completely absorbed within 3-4 weeks. SMI-SPON is biocompatible, non-toxic and non-pyrogenic. SMI-SPON is gamma sterilized and individually packed in ready to use blister packaging.

### Where to use

- Dental Surgery
- Plastic Surgery
- Vascular Surgery
- Orthopaedic Surgery
- ENT Surgery
- Neuro Surgery
- Abdominal Surgery
- Anorectal Surgery
- Genito Urinary Surgery
- Gynaecological Surgery

### How to use SMI-SPON

- Cut a piece to the desired size
- Apply SMI-SPON either in a dry state or saturated with physiological salt solution.
- If used dry, a piece of SMI-SPON is applied to the bleeding place and held in place with moderate pressure until hemostasis results.
- If used wet, first immerse in the solution and then withdraw, squeeze between gloved fingers to expel air bubbles, and the replace in saline until needed.
- When bleeding is controlled, SMI-SPON may be removed from the site or left in situ.



SMI-SPON Standard



SMI-SPON Dental



SMI-SPON Tampon



SMI-SPON Special

## Available products and references.

SMI-SPON Standard	80 x 50 x 10 mm	10 Pieces / box	REF: ZHG805010
SMI-SPON Dental	10 x 10 x 10 mm	32 Pieces / box	REF: ZHG101010
SMI-SPON Tampon	80 x 30 mm DIA	6 Pieces / box	REF: ZHG8030
SMI-SPON Special	80 x 50 x 1 mm	20 Pieces / box	REF: ZHG805001

[www.sutures.be](http://www.sutures.be)